


Important Safety Instructions

For the latest information of EyeSpecial C-II, visit our website at <http://www.shofu.com>. Select the region you purchased the camera.

Package contents

When first opening your EyeSpecial C-II package, please ensure that all the following components are present. If anything is missing, please contact your SHOFU dealer as soon as possible.

*Alkaline AA batteries are not included in EU countries and some other countries due to regulations.

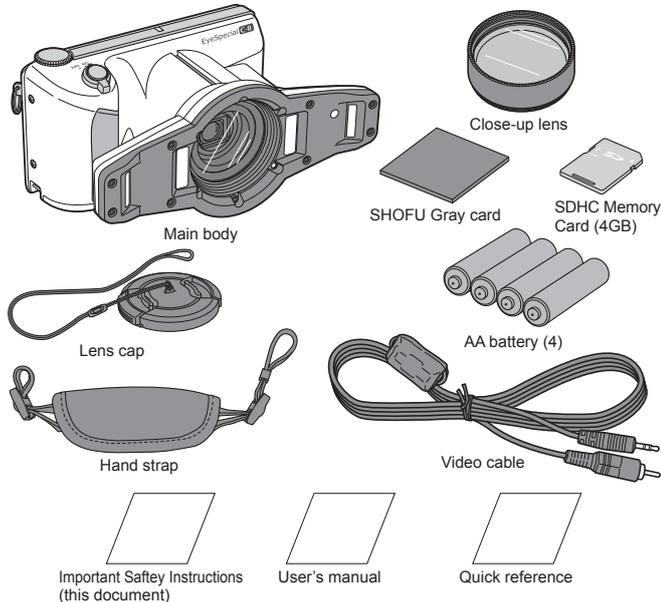

Optional accessories

Following optional items sold separately are necessary to use this camera.

- Recording media SD Card/SDHC Card (*Incompatible with SDXC Card)
- AA batteries (Alkaline battery / Rechargeable nickel hydride battery)

* If your computer does not have a SD/SDHC slot, please purchase a SD/SDHC card reader/writer to transfer data from your memory card to your computer.

Safety Information

Understand the following items to use the product safely.

Descriptions of warnings and cautions

This manual provides safety information using the following 2 symbols to prevent accidents resulting in injury or death. Understand the meaning of these symbols to operate the device safely.

 Warning	Warning indicates a potential hazardous situation which, if not avoided, could result in death or serious injury.
 Caution	Caution indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury or in property damage.

Other notes and signs

Other notes and signs used in this manual are shown below.

-  **Note** Indicates that potential malfunctions may occur if ignore the description
-  Supplementary explanation to facilitate your operation

For use

Warning

- **If the camera overheats, gives off burnt odor, generates smoke or in any way appears abnormal during use, immediately discontinue use and remove batteries. The batteries might be also overheated. Be careful to avoid burn when removing them.**
Ignoring these warnings and continuing use may result in electric shock, burn and spontaneous ignition. Immediately contact your SHOFU dealer for repair.
- **Should the internal components of the camera become exposed by dropping or damaging the unit, immediately discontinue use and remove batteries while being careful not to touch the exposed parts.**
The flash is high voltage and might cause electrical shock. Ignoring these warnings and continuing use may cause electrical shock, burn or fire. Please contact your SHOFU dealer for servicing instructions.
- **Keep out of reach of children.**
Exercise extreme caution when using this camera near children to avoid accidents or injuries.

- **Never disassemble, attempt repair or overhaul the camera.**
There are some high voltage parts in this camera. Touching internal components might cause electric shock. Any manipulation might also cause malfunction of the camera, resulting in injuries, electric shock or fire. Please contact your SHOFU dealer when repair or overhaul is necessary.
- **Use only the designated battery. Avoid using damaged or deformed batteries.**
Using any defective battery may cause fluid leak, fire or explosion.
- **Do not trigger the flash in close proximity to human eyes. Use eye protection when using the flash close to human eyes.**
Exposure to the intense light produced by the flash may injure eyesight.
- **If any foreign object (metal, liquid or powder) gets inside the camera, immediately discontinue use and remove the batteries from the unit.**
Continued use of the camera could result in fire or electrical shock. Consult the dealer from who you bought the product for further assistance.
- **Do not use this camera in a narrow space, such as in a bag, box or drawer.**
This may generate heat or cause fire due to insufficient heat release.
- **Do not use this camera in places where it gets wet easily, including near a washbasins, sink, water tank or anywhere you use water. Do not place anything containing liquid on the camera.**
This may cause fire or electric shock.
- **Do not use this camera in places such as: a place filled with highly combustible gas or a place where gasoline, benzene or thinner is stored.**
This might cause explosion or fire.

Caution

- **When using this camera for a long period of time, avoid keeping contact with skin.**
The camera body becomes hot and might cause cryogenic burn.
- **Do not remove the batteries and SD card immediately after a long period of continuous use.**
The batteries and SD card can become hot during use. Touching hot batteries and card might cause skin burn. Turn off the camera and wait until it cools down.
- **Do not use the flash while covering it with fingers or some object.**
The flash generates heat and might cause burn.
- **Do not press liquid crystal display too strongly or apply strong force unnecessarily.**
Doing so might break the monitor, resulting in injuries. If the liquid inside comes in contact with skin, immediately rinse with water. In case of contact with eyes, immediately flush with water and seek medical advice.
- **Handle the camera and accessories gently.**
Failure to do so might cause injuries.
- **Always use the hand strap to avoid injuries caused by dropping the unit.**
- **The hand strap must be checked periodically to find loosening or fraying of the string if it occurs.**
Ignoring any abnormalities and continuing to use the camera may result in injuries.

Maintenance and storage

Warning

- **Small accessories should be kept out of reach of children to avoid accidental swallowing.**
If a child swallows any part of this product, immediately seek medical advice.
- **Do not use combustible solutions such as benzene or thinner to clean this camera.**
This might cause explosion or fire.

Caution

- **Avoid using and storing this camera (including batteries) under direct sunlight, high temperature and high humidity or near fire. It should also avoid being used or stored in the places where corrosive gas generates or near a machine that generates strong magnetism or electromagnetic wave.**
This might cause defect, deformation of the unit, leakage of the battery liquid, heat generation, explosion, fire or injuries.

 **Manufacturer**
SHOFU INC. Kyoto 605-0983, Japan

SHOFU DENTAL CORPORATION San Marcos, CA, USA

 SHOFU DENTAL GmbH Ratingen, Germany

SHOFU DENTAL ASIA-PACIFIC Singapore

SHOFU DENTAL TRADING (SHANGHAI) China