Hybrid Points Kit

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

[EC REP]

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

04.02020-09
72413

READ CAREFULLY BEFORE USE

de

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en

PRODUCT DESCRIPTION

SHOFU Hybrid Points, diamond-coated abrasives are used for cavity, crown and bridge preparation.

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds described in Table A. •Insert the shank completely into the chuck of the handpiece. •Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. •Perform test run to check safety and smooth operation without run-out prior to use.
- Use intermittently with gentle touch and water spray. •Avoid any use at acute angles or with excessive pressure. •Use local dust extractor and protective equipment while using. •This product is intended for use by dental professionals only. •Do not use this product for any purposes other than the intended use. •Dispose in accordance with relevant local and national regulations.

STORAGE AND STERILIZATION

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. •Perform autoclave under the condition described in Table B. •When using intraorally, sterilize before each use. •For reuse after using, remove any contaminants immediately with washer disinfector, then sterilize. •Follow the instructions for use of washer disinfector and sterilizer, respectively. •Do not use chlorine disinfectant liquid. •For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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