# GlasIonomer Cement CX-Plus

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## en READ CAREFULLY BEFORE USE

### Glass polyalkenoate cement for cementing

Glaslonomer Cement CX-Plus is a radiopaque glass ionomer luting cement.

#### INDICATIONS

Cementation of crowns, bridges, and inlays

#### PRECAUTIONS

- Do not use this product on patients with known allergies to this material and/or glass polyalkenoate cements.
- Operators with known allergies to this material and/or glass polyalkenoate cements must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- The liquid and mixture of this product should not come in contact with intraoral soft tissue, skin or eyes. In case of accidental contact with intraoral soft tissue or skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- This product is intended for use by dental professionals only.

#### DIRECTIONS FOR USE

- Cleaning of cavity and abutment tooth  
Thoroughly remove the temporary sealing or cement. Rinse with water and dry.  
➤ In the case where pulp is exposed or the affected area is close to the pulp, cap the pulp with calcium hydroxide etc.
- Dispensing of Powder and Liquid (standard powder/liquid ratio: 2.0 g/1.0 g)  
Dispense one level scoop of Powder with the measuring scoop provided and two drops of Liquid separately.  
➤ Always use the measuring scoop provided and level off the powder for accurate dispensing.  
➤ Invert the liquid bottle prior to use to dispense drops of bubble-free liquid.  
➤ Remove any liquid plugging the opening of the nozzle with wet gauze etc. for accurate measuring.  
➤ Tightly close the bottle caps (powder and liquid) immediately after each use.  
➤ Use extra caution to keep the powder away from any moisture.
- Mixing  
Mix dispensed powder and liquid all together for 30 sec. until it becomes homogeneous paste.

Mixing time (23±1 °C/73.4±1.8 °F)	30"	
Working time (23 °C/73.4 °F)	3' 30"	From start of mixing
Setting time (37 °C/98.6 °F)	4' 00"	From end of mixing

- Cementation  
Clean the surfaces of restoration and natural tooth to be cemented and dry. Apply the mixture to cement the restoration following the conventional method.  
(After placing the restoration, keep it away from saliva contamination for around 10 min.)  
➤ Remove excess material quickly with wet gauze etc before it gets hardened.

#### NOTE ON USE

- Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container.  
(Example: YYYY-MM-DD → Year-Month-Date of the expiration date)

#### STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high humidity and keep away from direct sunlight.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary: [https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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