## English

### PROPHYLAXIS PASTE WITH FLUORIDE

MERSAGE paste containing fluoride removes plaque and provides a smooth and polished tooth surface.

#### INDICATIONS

	Regular	Fine	Plus
1. Removal of plaque and extrinsic stains on the tooth surfaces, and polishing the tooth surfaces	✓	✓	
2. Polishing upon removal of orthodontic appliances	✓	✓	
3. Polishing after scaling		✓	✓
4. Polishing after bleaching		✓	✓
5. Prevention of reattached stain			✓
6. Cleaning of restorative materials and polishing the surface			✓

#### DIRECTIONS FOR USE

1. Remove cap and extrude slowly to avoid waste. Replace cap tightly at once.
2. Apply a small quantity of paste to a brush or a cup. Do not mix with water.
3. Apply paste to all tooth surfaces using low speed (less than 2,000 min<sup>-1</sup>). Do not operate with water spray.
4. Allow patient to rinse completely after polishing is completed.

#### CAUTION

1. Do not use on patients with allergies.
2. Stop usage if a patient experiences unusual or excessive sensitivity and refer for medical attention.
3. It is recommended that the operators and the patient wear eye protection. Rinse with water should paste come in contact with the eyes.
4. Use this product for its indicated purpose only.

#### STORAGE

Store tightly capped at room temperature or below. Should liquid separation or drying out occur, this will not reduce the effectiveness of the product.

#### INGREDIENTS

Regular	Pumice, Binders, Surfactants, Flavoring, Colorings, Sodium Monofluorophosphate ( 0.34%)
Fine	Calcium Carbonate, Silica, Binders, Surfactants, Flavoring, Coloring, Sodium Monofluorophosphate ( 0.34%)
Plus	Silica, Binders, Surfactants, Flavoring, Colorings, Sodium fluoride ( 0.21%)

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