VINTAGE LD

05.02.2020-09
72475

en Instructions for use	de Gebrauchsanleitung
fr Mode d'emploi	nl Gebruksaanwijzing
es Instrucciones de uso	it Istruzioni per l'uso
ru Инструкция по применению	pl Instrukcja obsługi
ro Instructiuni de utilizare	sv Bruksanvisning
el Οδηγίες χρήσης	pt Instruções de uso

Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukui, Fugashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos, California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

EC REP
SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

en Lithium Disilicate Bond Porcelain

READ CAREFULLY BEFORE USE

VINTAGE LD is a dental ceramic to be layered over Lithium disilicate press glass ceramic, VINTAGE LD PRESS. Body shades are available in classical shades as well as a variety of effect, translucent and opalescent incisal shades. VINTAGE LD is a low fusing porcelain and can be fired at 800 °C or lower.

INDICATIONS

Build up porcelain for fabrication of all ceramic restorations including lithium disilicate

PRECAUTIONS

- If any inflammation or other allergic reactions occur on operator, immediately discontinue use and seek medical advice.
- Avoid contact with eyes. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Wear protective glasses etc. while grinding and polishing this product to avoid damages to eyes.
- Use local dust extractor, dust protective mask, etc. while grinding this product to avoid harmful influence of the dust on the human body.
- This product should not be used for the cases where residual abutment tooth is too small, or for the patients with bruxism or occlusal abnormality etc.
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Refer to the separate instruction manual or step card.

NOTE ON USE

- Final form and size should be restored with body porcelain to build up opalescent enamel or translucent porcelain accurately.
- Drying time should be altered depending on the size of the restoration to be fired. (Restorations might explode inside the furnace if moisture remains inside.)
- Enamel porcelain should be built up oversized in consideration of firing shrinkage.
- When building up additional porcelain onto the dried area, wet the area with water moistened brush in advance.
- Optimum firing schedule is different depending on the type and model of the furnace. Try to determine the optimal conditions at which every restoration is fired completely.
- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- Do not use this product in conjunction with PFM's, metal frames or ceramic frames (alumina).
- Do not use this product for any purpose other than specifically outlined in INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container. (Example YYYY-MM-DD → Year-Month-Date of the expiration date)

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high temperature and high humidity. Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

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