# BeautiCem Try-in Paste

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## Try-in Paste for Resin Cement

### PRODUCT DESCRIPTION

BeautiCem Try-in Paste is a trial paste to help in the selection of the suitable shade of resin cement, BeautiCem Veneer, before placing the restoration. It is water-soluble and can be easily removed after confirming the shade.

### INDICATIONS FOR USE

Confirmation of the suitable cement shade before placing the indirect restoration

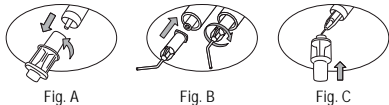
### PRECAUTIONS AND WARNINGS

- Do not use this product on patients with known allergies to this material.
- Operators with known allergies to this material should not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- Use protective medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, flush immediately with plenty of water and seek medical advice.
- Follow the instructions for use of each dental material, instrument or equipment to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package. (Example □ YYYY-MM-DD → Year-Month-Date of the expiration date)
- This product is intended for use by dental professionals only.

### DIRECTIONS FOR USE

- Preparation of the syringe  
When storing these materials in a refrigerator, allow them to reach room temperature before use. Attach the needle tip to the syringe as indicated in the figures below.

- 1-1 Pull off the shield cap from the syringe while turning it slowly. (Fig. A)
  - 1-2 Attach the needle tip by turning it clockwise until it is securely attached. (Fig. B)
  - 1-3 Insert the needle tip into the small hole on the head of the shield cap immediately before use to protect the material from ambient light. (Fig. C)
- When replacing the needle tip, always use clean medical (dental) gloves.

- Preparation of the indirect restoration  
Confirm the adhesive surface of the restoration is treated properly according to the manufacturer's instructions (eg. Alumina sand blasted etc).
- Try-in the indirect restoration  
Select the proper shade of BeautiCem Try-in Paste and apply to the adhesive surface of the restoration. Place the restoration onto the tooth to confirm the shade.  
After try-in, thoroughly rinse off the Try-in Paste remaining on the tooth and restoration surface.  
In case of shade mismatch, try with another shade of Try-in Paste until you obtain the optimal shade.  
➢ When unused for a long period of time, Try-in Paste may separate at the tip of the syringe. Remove any separated material before use.
- Cementation of the indirect restoration  
After determining the optimal shade, use BeautiCem Veneer of the same shade to cement the restoration.

### COMPOSITION

Polyethylene glycol dimethacrylate, polyethylene glycol, viscosity improver, pigments and others

### SHADES

L-Value (Low Value), M-Value (Medium Value), H-Value (High Value), Ivory-L (Ivory-Light), Ivory-D (Ivory-Dark)

### STORAGE

Store at room temperature (1-30 °C/ 34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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