CE 0123

SHOFU INC.
11 Kamitakanami-cho, Fukukawa,
Higashiyama-ku, Kyoto 605-0833, Japan
www.shofu.com

SHOFU DENTAL GmbH
Am Brill 17, 40378 Ratingen, Germany
SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

03/2019-03

71805

VINTAGE Art LF

RU Инструкция по применению
PL Instrukcja obsługi
RO Instrucția de utilizare
EL Οδηγίες Xρήσης
PT Instruções de uso
ES Instrucciones de uso
IT Istruzioni per l'uso

EN Low-fusing porcelain stain

VINTAGE Art LF are fluorescent porcelain stains for shade adjustment and characterization. They are low-fuse type and can be fired at 800 °C or lower. VINTAGE Art LF can be used to adjust the shade of VINTAGE LD PRESS, VINTAGE LD, VINTAGE ZR, zirconia and other porcelain materials.

COLOR

O, O-Br, BI, Br, Y, CY, V, G, B, W, P, B-Br, BI-G, Vn, K, DR-Br, WR, RP, Gr, AS, BS, CS, DS, GP

INDICATIONS

Shade adjustment of Lithium disilicate and ceramic restorations

PRECAUTIONS

- If any inflammation or other allergic reactions occur on the operator, immediately discontinue use and seek medical advice.
- Avoid contact with skin and eyes. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Wear protective glasses etc while grinding and polishing this product to avoid damages to eyes.
- Use local dust extractor, dust protective mask, etc while grinding this product to avoid harmful influence of the dust on the human body.
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Cleaning of the restoration
Clean the restoration throughly with water or alcohol etc to remove any dirt and other contaminants.
- Preparation
Dispense the desired paste onto a paper pad or stain palette. The pastes can be mixed at any ratio to obtain the desired shade. The viscosity can be adjusted by mixing with VINTAGE Art LF STAIN LIQUID.
- Application
Apply the paste with a brush.
- Firing
Example of firing schedule

Firing	Preliminary dry Preheat	Firing Schedule	Holding
First firing		4 min.	
Second firing		450-(A)-750 °C	1 min.

[A:firing in the air]
[Heat rate: 55 °C/min]

NOTE ON USE

- Tightly close the cap immediately after each use.
- Dispense necessary amount of paste and VINTAGE Art LF STAIN LIQUID. Do not put remaining material back into the container.
- Do not touch the material with bare hands.
- Use the dispensed material immediately after dispensing.
- Optimal firing conditions vary due to the different designs and operating voltages of the porcelain furnaces. It is essential to carry out test firings before firing actual restorations.
- Do not use any liquid other than the VINTAGE Art LF STAIN LIQUID.
- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- Avoid any source of ignition since this product is flammable.
- Do not mix with any other products.
- Do not use this product for any purpose other than specifically outlined in INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container.
(Example YYYY-MM-DD → Year-Month-Date of the expiration date)

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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