P.R.E.S.A.G.E.

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FR Mode d'emploi  
IT Istruzioni per l'uso

02-2013-12  
AHL000077  
SD1009/2

Manufacturer  
SHOFU INC.  
11 Kanitakamatsi-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983,  
Japan  
EC REP  
SHOFU DENTAL GmbH  
Am Brühl 17, 46878 Ratingen,  
Germany

SHOFUDENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059,  
USA  
SHOFUDENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

## EN NON-FLUORIDE PROPHYLAXIS PASTE

PRESSAGE paste is designed for cleaning tooth surfaces before bleaching and application of etchants, bonding agents and sealants. It is fluoride free since fluoride may adversely affect bleaching efficacy and adhesion. It provides a cleaned tooth surface safely.

### COMPOSITION

Pumice Powder, Sodium Carboxymethyl Cellulose, Polyethylene Glycol, Glycerin, Sodium Laurylsulfate, Sorbitol

### INDICATIONS

- Cleaning before bleaching
- Cleaning before cavity preparation
- Cleaning before fissure sealing
- Cleaning before fixing orthodontic appliances

### PRECAUTION

- Do not use on patients with allergies.
- Stop usage if a patient experiences unusual or excessive sensitivity and refer for a medical attention.
- It is recommended that the operators and the patients wear eye protection. Rinse with water should paste come in contact with the eyes.
- Use this product for its indicated purpose only.

### DIRECTIONS FOR USE

- Remove cap and extrude slowly to avoid waste. Replace cap tightly at once.
- Apply a small quantity of paste to brush.
  - Do not mix with water.
- Apply paste to tooth surfaces using low speed (less than 2,000 min<sup>-1</sup>).
  - Do not operate with water spray.
- Allow patient to rinse completely after cleaning is completed.
  - If the paste remains on the tooth surfaces, it might reduce the effectiveness of bleaching and adhesion.

### STORAGE

Store tightly capped at room temperature or below. Should liquid separation or drying out occur, this will not reduce the effectiveness of the product.

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