



Veracia Veracia SA

EN Instructions for use

02:2014-05
W000001



Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan

EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos, California
92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684



EN Micro filled hybrid composite resin teeth

Veracia/Veracia SA are micro filled hybrid resin teeth that provide excellent wear resistance and impact resistance. Veracia/Veracia SA are available in 17 shades.

SHADES

W1, A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4

COMPOSITION

Enamel / Dentin : UDMA, PMMA, Micro fumed silica, others
Base : PMMA, others

INDICATIONS

Used as denture teeth to produce partial or full dentures

PRECAUTIONS

1. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
2. Use a dust extractor, dust protective mask etc. while cutting or polishing this product, to avoid harmful effects of dust on the human body.
3. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
4. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Produce full or partial dentures following the conventional method.

- Carefully contour and polish this product since its enamel and dentin layers are hard and highly wear resistant.
- Do not use strong alkaline gypsum dissolving agent (pH 10 or over) to avoid discoloration.
- Do not expose to fire when fabricating wax denture.
- Give mechanical retention when using thermoplastic denture base resins. Veracia does not bond to thermoplastic denture base resins.
- Give the following general instructions to patient:
 - a. Keep the denture clean. Gently remove food debris with a soft denture brush after eating. Using denture cleanser is more effective.
 - b. When the denture gets dirty or accumulates plaque, clean by using a denture brush and dentifrice.

STORAGE

Avoid high temperature and high humidity. Keep away from direct sunlight.

MOULDS

Veracia	Maxillary/ Mandibular	Shape	Size
Anterior	Maxillary	O LO S LS ST LT	4* 5 6
	Mandibular	NA A LA	4* 5 6
Posterior	Maxillary Mandibular	S M L	

* Maxillary size 4 is not available in LO, LS and LT shapes.

* Mandibular size 4 is not available in LA shape.

Veracia SA	Maxillary/ Mandibular	Shape	Size
Anterior	Maxillary	S O ST	4 5 6
	Mandibular	MA	4 5 6 7
Posterior	Maxillary Mandibular	S	28 30 32 34

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.