

Veracia SA ANTERIOR & POSTERIOR



**EXCELLENT AESTHETICS
CONSIDERABLE TIME-SAVING**




A VISION BECOMES REALITY

Age based and functional prosthetics with high wear comfort

With the development of the new semi-anatomical **Veracia SA** anterior and posterior teeth the wear on the remaining natural teeth was considered for the first time.

The development is based on the fully anatomical Veracia tooth line. The functional characteristics of the Veracia teeth were preserved while precisely predetermined sectors were provided with abrasion zones. The result is substantially improved masticatory performance leading to increased stability during lateral excursion, resulting in improved wear comfort with reduced pressure to the soft tissues and less stress to the alveolar bone and TMJ.

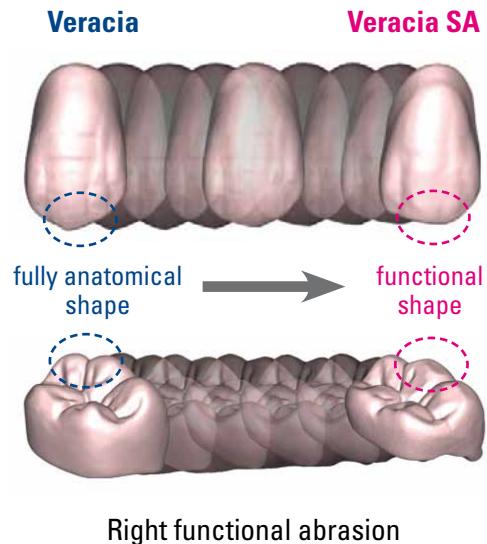

Setup for all common occlusion concepts

With **Veracia SA**, you are not bound to any occlusion concept. The posterior teeth can be easily and precisely set up antagonistically due to the clear occlusal structure, the effective central occlusion and the systematically placed abrasion zones.

- Balanced occlusion
- Lingualised occlusion
- Tooth-to-tooth occlusion
- Tooth-to-two-teeth occlusion




Balanced Side



Working Side



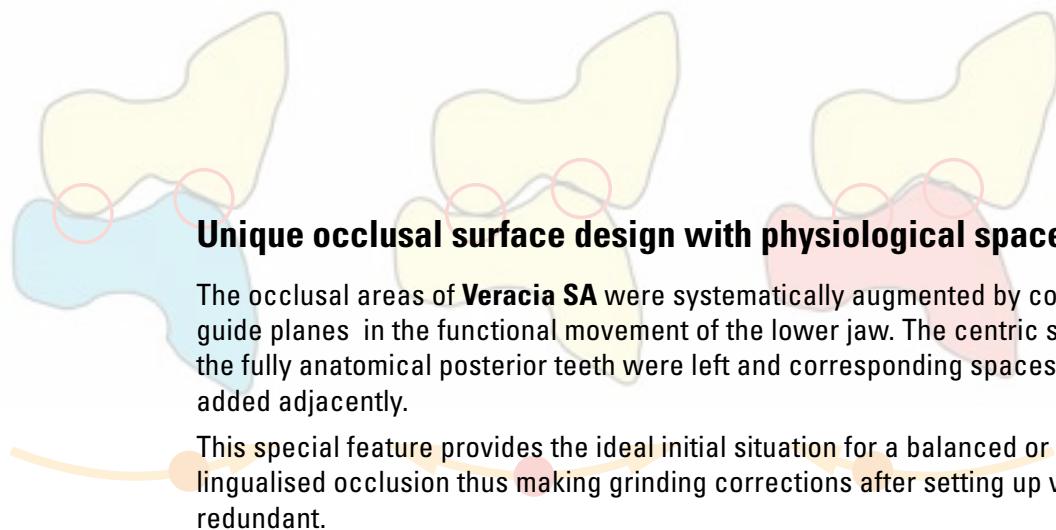


Unique occlusal surface design with physiological spaces


The occlusal areas of **Veracia SA** were systematically augmented by concave guide planes in the functional movement of the lower jaw. The centric stops of the fully anatomical posterior teeth were left and corresponding spaces were added adjacently.

This special feature provides the ideal initial situation for a balanced or linguinalised occlusion thus making grinding corrections after setting up virtually redundant.

Beyond this, these spaces make it easy for patients to move into static occlusion so that lateral shear forces are kept to a minimum.

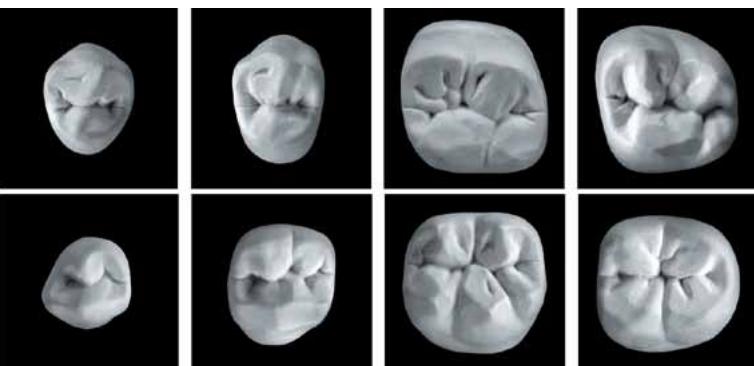


Deep occlusal fissures combined with wear facets for specific movements provide simple and unmistakable alignment during the setting up, along with a more effective masticatory performance of the finished denture.

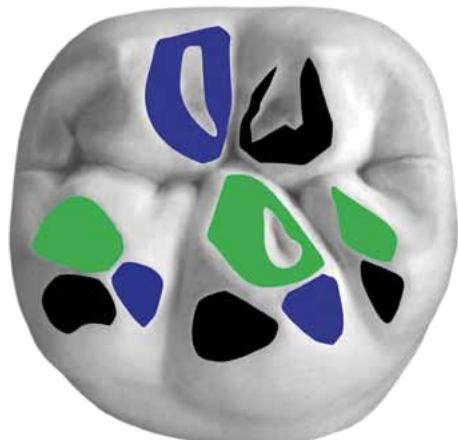

- Laterotrusion
- Mediotrusion
- Protrusion

Teeth that find their way to one another

Veracia SA supports your efficiency during your daily fabrication of dentures. Due to the unique functional design the setting up of the posterior teeth virtually happens by itself.

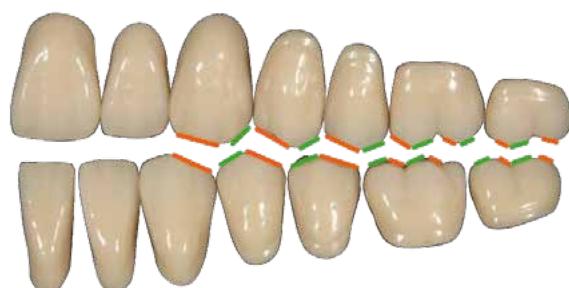

ONE, TWO, Q³ PACK ... DONE !

Excellent aesthetics and function with considerable time-saving

The physiological design of **Veracia SA** posterior teeth enabled the development of a unique wax-free assortment holder with an integrated set-up support – the **Q³ PACK**.

With the **Q³ PACK** you achieve, for the first time, an effortless set-up of the posterior teeth with consistant excellent function and user reliability, independent of the articulator system used.



Q³ PACK reduces the setting up time of eight **Veracia SA** posterior teeth to a few minutes. Time-consuming preparations or complex equipment are not necessary.

Q³ PACK is precise and provides the correct contact relation for **Veracia SA** posterior teeth. Functional grinding corrections are kept to a minimum.

Q³ PACK contains single posterior teeth, allowing individual adjustments.

Only 3 steps to a functional setup



The anterior teeth are set-up and the flanges are prepared as usual. Place the mesial recess of Q³ PACK on the canine tooth and align it distally to the alveolar line.



Then check the correct position of the grips in horizontal alignment with the occlusal plane and wax the teeth in.




A little bit of pressure on the grip releases the set-up.



Finally, close the interdental spaces with wax – done.

Outstanding physical properties with MF-H composite technology

Veracia SA teeth consist of MF-H (microfilled hybrid) composite reinforced with layered glass just like the proven fully anatomical teeth of the Veracia tooth line. This homogenous structure achieves excellent physical properties and outstanding consistency.

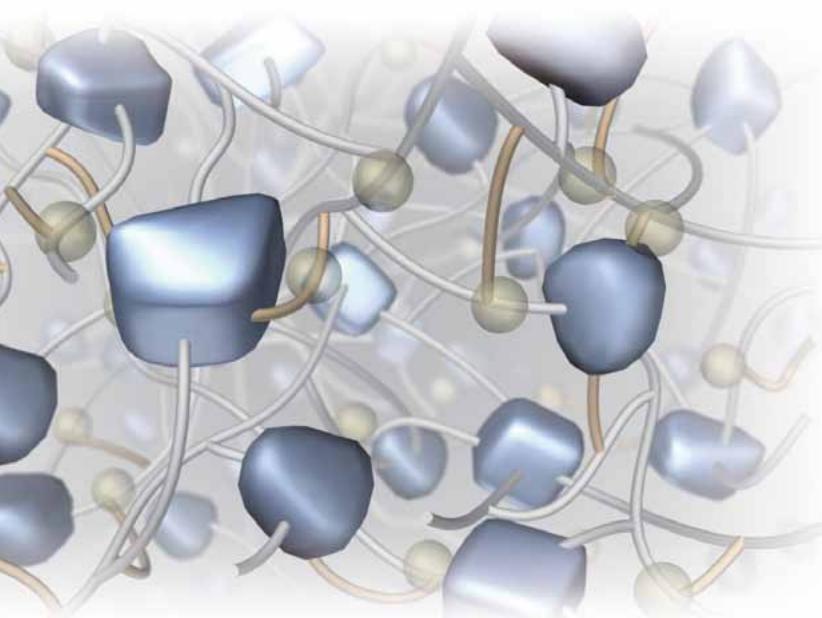

SIMPLE

No complex equipment

EFFICIENT

Wax-free assortment = Set-up support

FAST

No time-consuming preparations

PRECISE

Always correct contact relationships

INDEPENDENT

Useable with any articulator system

Why Veracia SA?

- Age based and functional teeth
- Not bound to any occlusion concept
- Saves you set-up time
- Balanced occlusion just by setting up
- High level of user reliability
- Efficient fabrication in the dental laboratory
- Constant high quality of dentures

The new way of teeth set-up

With **Q³ PACK** you immediately save valuable time when setting up dentures.



MDT German Bär in Sankt Augustin, Germany prepared the restorations and photos.

Quick Quality Quartet



Veracia SA ANTERIOR



Just select

A state-of-the-art computer-supported manufacturing technique guarantees the consistent symmetry of the left and right anterior and posterior teeth as well as a true-to-dimension size allocation.



SQUARE

	43.0	
S4	9.2	
	7.9	
	45.7	
S5	9.8	
	8.4	
	48.5	
S6	10.4	
	8.9	



SQUARE-TAPERING

	43.5	
ST4	9.6	
	8.0	
	46.3	
ST5	10.2	
	8.5	
	49.0	
ST6	10.8	
	9.0	




OVOID

	43.4	
04	10.4	
	8.0	
	46.2	
05	11.0	
	8.5	
	49.0	
06	11.7	
	9.0	



MORE ABRASIVE

	32.5	
MA4	9.4	
	4.7	
	34.6	
MA5	10.1	
	5.0	
	36.7	
MA6	10.7	
	5.3	
	38.8	
MA7	11.3	
	5.6	

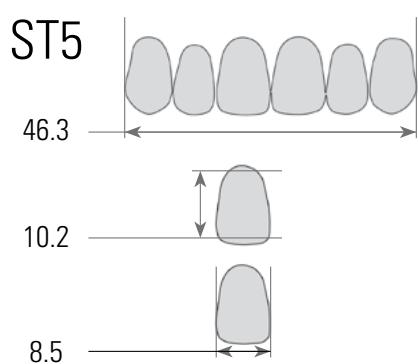

The anterior and posterior teeth are available in the VITA* Classical shades A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4 and in the SHOFU whitening shade W1.



Q³ PACK is offered in the shades A2, A3 and A3.5 .

Veracia SA POSTERIOR & Q³ PACK


Veracia SA posterior teeth are alternatively available as conventional set or as wax-free Q³ PACK.

S28	28.0 8.7 30.0 8.9		Veracia SA POSTERIOR and Q ³ PACK
S30	30.0 9.4 32.5 9.5		Veracia SA POSTERIOR and Q ³ PACK
S32	32.0 10.0 34.5 9.9		Veracia SA POSTERIOR Q ³ PACK
S34	34.0 10.6 36.0 10.7		Veracia SA POSTERIOR Q ³ PACK

Recommendations for upper / lower anterior teeth

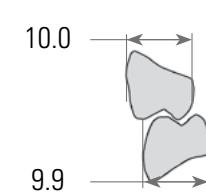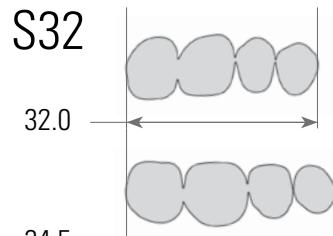

Overbite

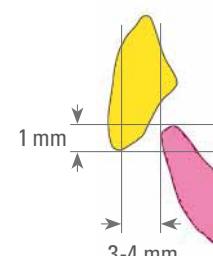


Upper
Lower

S4 ST4 04	S5 ST5 05	S6 ST6 06
-----------------	-----------------	-----------------

Upper
Lower

MA4	MA5	MA6
-----	-----	-----

Overbite

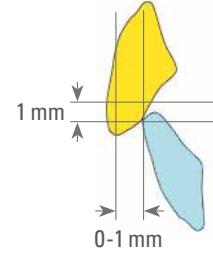

Upper
Lower

S4 ST4 04	S5 ST5 05	S6 ST6 06
-----------------	-----------------	-----------------

MA5	MA6	MA7
-----	-----	-----

All dimensions in millimetres

* VITA is a registered trademark of VITA Zahnfabrik, Bad Säckingen

SYSTEM COMPONENTS



Veracia SA ANTERIOR

3 upper jaw tooth shapes in sizes 4, 5 and 6

- S (Square)
- ST (Square-Tapering)
- O (Ovoid)

1 lower jaw tooth shape in sizes 4, 5, 6 and 7

- MA (More Abrasive)



Veracia SA POSTERIOR

1 tooth shape upper jaw / lower jaw in sizes S28, S30, S32, S34



Veracia SA POSTERIOR Q³ PACK

1 tooth shape upper jaw / lower jaw in sizes S28, S30, S32, S34



Veracia SA 28 TEETH SET A2 / A3

1 anterior teeth set upper jaw (ST6)

1 anterior teeth set lower jaw (MA6)

1 each Q³ PACK posterior teeth set (upper jaw / lower jaw, size 32)

PN: A2ST66 / A3ST66



Veracia SA AP FULL SET

13 anterior teeth sets (9 upper jaw / 4 lower jaw)

6 posterior teeth sets

(S30, S32, S34: 3 upper jaw / 3 lower jaw)

PN: FSA3



SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan (www.shofu.co.jp)

SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12 The Alpha, Science Park II, Singapore 117684 (www.shofu.com.sg)

SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China (www.shofu.com.cn)

SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92078-4059, USA (www.shofu.com)

SHOFU UK Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK (www.shofu.co.uk)

SHOFU DENTAL GmbH [EC REP] Am Brüll 17, 40878 Ratingen, Germany (www.shofu.de)

SHOFU is a registered trademark of SHOFU INC. All other trademarks and registered trademarks are the property of their respective holders. SHOFU INC. reserves the right to change specifications without notice.