


EyeSpecial C-III

Important Safety Instructions

 For the latest information on the EyeSpecial C-III visit our website at www.shofu.de.

Package contents

When first opening your EyeSpecial C-III package, please ensure that all the following components are present. If anything is missing, please contact your SHOFU dealer as soon as possible.

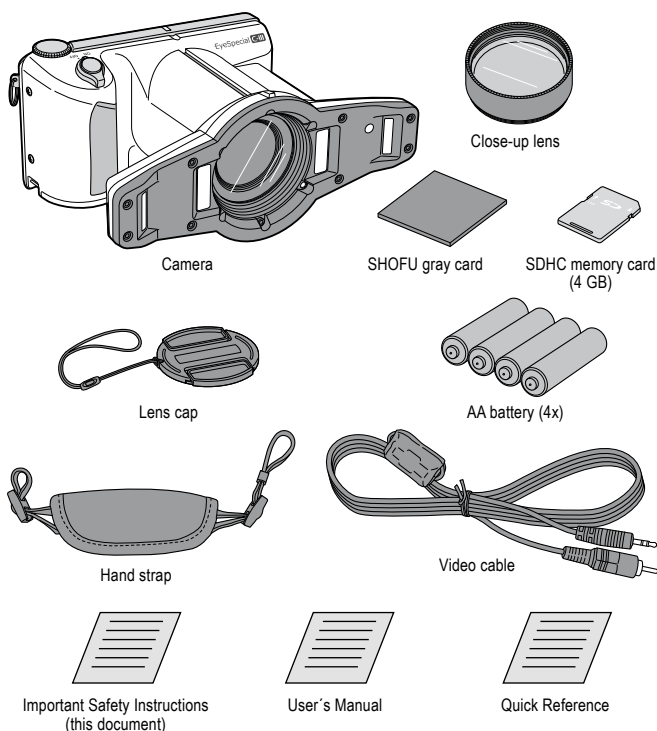

Optional accessories

Following optional items sold separately are necessary to use this camera:

- Recording media: SD Card/SDHC Card* (incompatible with SDXC cards)
- AA batteries (alkaline batteries / rechargeable nickel hydride batteries (NiMH))



* If your computer does not have a SD/SDHC slot, please purchase a SD/SDHC card reader/writer to transfer data from your memory card to your computer.

Safety Information

Please read the following information carefully for a safe use of the product.



Description of "Warning" and "Caution" symbols

This manual provides safety information using the following 2 symbols to prevent accidents resulting in injury or death. Understand the meaning of these symbols to operate the device safely.

 Warning	Warning indicates a potential hazardous situation which, if not avoided, could result in death or serious injuries.
 Caution	Caution indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injuries or in property damages.

Other notes and signs

Other notes and signs used in this manual are shown below:

-  **Note** Indicates that potential malfunctions may occur if you ignore the description.
-  **Info** Supplementary explanation to facilitate your operation.

During usage

Warning

- **If the camera overheats, gives off burnt odor, generates smoke or in any way appears abnormal during use, immediately discontinue use and remove the batteries. The batteries might also be overheated. Be careful to avoid burning when removing them.**
Ignoring these warnings and continuing use may result in electric shock, burn and spontaneous ignition. Immediately contact your SHOFU dealer for repair.
- **Should the internal components of the camera become exposed by dropping or damaging it, immediately discontinue use and remove the batteries. Be careful not to touch the exposed parts.**
The flash is high voltage and might cause electrical shock. Ignoring these warnings and continuing use may cause electrical shock, burn or fire. Please contact your SHOFU dealer for servicing instructions.
- **Keep out of reach of children.**
Exercise extreme caution when using this camera near children to avoid accidents or injuries.
- **Never disassemble, attempt repairing or overhauling the camera.**
There are some high voltage parts in this camera. Touching internal components might cause electric shock. Any manipulation might also cause malfunction of the camera, resulting in injuries, electric shock or fire. Please contact your SHOFU dealer when repair or overhaul is necessary.

- **Only use the designated batteries. Avoid using damaged or deformed batteries.**
Using any defective battery may cause fluid leak, fire or explosion.
- **Do not trigger the flash in close proximity to human eyes. Use protective eyewear when using the flash close to human eyes.**
Exposure to the intense light produced by the flash may injure eyesight.
- **If any foreign object (metal, liquid or powder/dust) gets inside the camera, immediately discontinue use and remove the batteries from the device.**
Continued use of the camera could result in fire or electrical shock. Consult the dealer from whom you bought the product for further assistance.
- **Do not use this camera in a narrow space, such as in a bag, box or a drawer.**
This may generate heat or cause fire due to insufficient heat release.
- **Do not use this camera in places where it could easily get wet, like near washbasins, sinks, water containers or anywhere you use water. Do not place anything on the camera that contains liquid.**
This may cause fire or electric shock.
- **Do not use this camera in places that are filled with highly combustible gas or in places where gasoline, benzene or thinner are stored.**
This might cause explosion or fire.

Caution

- **When using this camera for a long period of time, avoid skin contact.**
The camera body becomes hot and might cause skin burn.
- **Do not remove the batteries and SD memory card immediately after a long period of continuous use.**
The batteries and the SD memory card can become hot during use. Touching hot batteries and cards might cause skin burn. Turn off the camera and wait until it has cooled down.
- **Do not use the flash while covering it with fingers or any object.**
The flash generates heat and might cause burn.
- **Do not press the liquid crystal display too strongly or apply strong force unnecessarily.**
Doing so might break the monitor, resulting in injuries. If the liquid inside comes in contact with your skin, immediately rinse with water. In case of contact with your eyes, immediately flush with water and seek medical advice.
- **Handle the camera and accessories gently.**
Failure to do so might cause injuries.
- **Always use the hand strap to avoid injuries caused by dropping the camera.**
- **The hand strap must be checked periodically to detect possible loosening or fraying of the strap.**
- **Ignoring any anomalies and continuing to use the camera may result in injuries.**

Maintenance and storage

Warning

- **Small accessories should be kept out of reach of children to avoid accidental swallowing.**
If a child swallows any part of this product, immediately seek medical advice.
- **Do not use combustible solvents such as benzene or thinner to clean this camera.**
This might cause explosion or fire.

Caution

- **Avoid using and storing this camera (including batteries) under direct sunlight, high temperature and high humidity or near fire. It should also be avoided to use or store the camera in places where corrosive gas generates or near machines that generate strong magnetism or electromagnetic waves.**
This might cause defects, deformation of the camera, leakage of the battery liquid, heat generation, explosion, fire or injuries.



Manufacturer

SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA

SHOFU UK Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK

SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China

SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen, Germany (www.shofu.de)