

# AcryPoint

For PMMA resins



The AcryPoint silicone polishers are designed for adjusting, finishing and polishing of acrylic dentures, all acrylic appliances and acrylic impression trays without the need to use additional polishing pastes.



## Indications

- The coarse dark grey AcryPoints are used for adjustments.
- The brown AcryPoints with medium grit size are designed for finishing and prepolishing.
- The light grey AcryPoints with fine grit size are recommended for polishing.



## BENEFITS

- Three coordinated polishing steps
- Optimal surface polishing of PMMA
- Exact match of abrasive grains and binder
- High gloss polishing without the need of an additional polishing agent
- Low vibration performance
- Low heat development



	Shape	BP1	PC2
	ISO No.	100	050
Dark grey	HP 6 pcs.	0426	
	HP 12 pcs.		0423
Brown	HP 6 pcs.	0427	
	HP 12 pcs.		0424
Light grey	HP 6 pcs.	0428	
	HP 12 pcs.		0425



**Min<sup>-1</sup>** Recom. 5,000-10,000 / Max. 15,000 / CP 3-5 N  
Do not use intraorally!

PN	Quantity	Product
0423-0425	12 pcs.	AcryPoint, shape PC2, HP shank
0426-0428	6 pcs.	AcryPoint, shape BP1, HP shank