EN Instructions for use

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SV Bruksanvisning

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01:2018-05
300030236


Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP


SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684


CE 0120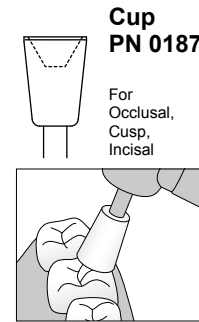

**Cup
PN 0187**
For
Occlusal,
Cusp,
Incisal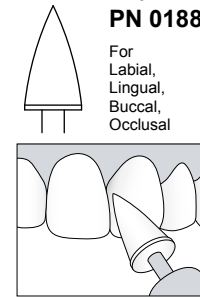

**Midi
PN 0188**
For
Labial,
Lingual,
Buccal,
Occlusal

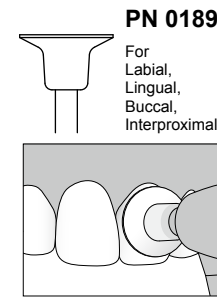**IC
PN 0189**
For
Labial,
Lingual,
Buccal,
Interproximal

Maximum permissible speed **15,000 min⁻¹**

EN

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

OneGloss is a one-step finisher and polisher that provides the ideal finish for composite restorations, simply by altering the contact pressure as you go from finishing to polishing.

INDICATIONS FOR USE

- Finishing and polishing of composite restoration
- Removal of excess resin or cement on tooth surface
- Removal of cements and polishing of enamel after removing and debonding orthodontic brackets
- Polishing of enamel after tooth scaling
- Removal of stain from tooth surface

PRECAUTIONS AND WARNINGS

- Do not use damaged or defective instrument, i.e. deformation, dirt, corrosion, exposure of shank due to repeated use.
- Use protective mask and eyewear when operating handpieces.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
- Apply heavy pressure for finishing and feather light pressure for polishing.

Shape	Application	Recommended pressure	
		Finishing	Polishing
Cup	Cusp, incisal, occlusal	1.0 N	0.3 N
Midi	Occlusal, labial, lingual, buccal		
IC	Labial, lingual, buccal, interproximal		

➢ Use at rotation speed of between 3,000 min⁻¹ to 10,000 min⁻¹ with intermittent water spray. Avoid exceeding maximum permissible speed, 15,000 min⁻¹.

➢ Use saliva ejector or suction to eliminate moisture and debris during the procedure.

- After using OneGloss M, remove any contaminants immediately, disinfect and sterilize.

STERILIZATION AND STORAGE

OneGloss M should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Avoid external pressure (physical load) and contamination.

When using intraorally, sterilize before use. Remove any contaminants using suitable cleaning liquid or disinfectant and perform autoclave.

Product name	Autoclave
OneGloss M	134 °C - 3'00" 121 °C - 30'00"

Follow manufacturer's instructions for use of cleaning liquid, disinfectant or sterilizer. Chloride disinfectant should be avoided.

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