GIC Polishing Kit

03:2019-03
71408

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
Am Brühl 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

CE 0123

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

T&F Hybrid Points	450,000 min ⁻¹
Dura-White Stones	30,000 min ⁻¹
CompoSite	20,000 min ⁻¹
Ceramisté	30,000 min ⁻¹

AUTOClave (Table B)

Temperature	Time
134 °C/273 °F	3'00"
121 °C/250 °F	30'00"

DE MAXIMAL ZULÄSSIGE DREHZAHL
FR VITESSE MAXIMALE DE ROTATION AUTORISÉE
NL MAXIMAAL TOELAATBARE ROTATIESNELHEID
ES VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN
IT MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
SV HÖGSTA TILLÄTNA ROTATIONSHASTIGHET
PT VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

DE AUTOKLAV
FR AUTOCLAVE
NL AUTOCLAAF
ES AUTOCLAVE
IT AUTOCLAVE
SV AUTOKLAV
PT AUTOCLAVE

EN

READ CAREFULLY BEFORE USE

This kit is designed for finishing and polishing conventional glass-ionomer cements, resin modified glass-ionomer and compomer restorations. Follow the manufacturer's instruction for the timing of start to finish and/or polish and for the varnish protection, if the restorative requires.

1. Remove excess material with T&F Hybrid Points or Dura-White Stones.
2. If required, varnish preferably with SHOFU HI GLAZE
3. Finish & Contour with T&F Hybrid Points or Dura-White Stones.
4. For polishing, use CompoSite Polishers. For superpolishing, use Ceramisté Ultra II Polishers.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

STERILIZATION

Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(CAUTIONS)

- Never exceed the maximum permissible rotation speeds. ●Use FG with lower speed airtight handpiece. High speed air turbine units can break cups or points. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●Use T&F Hybrid Points with water spray. ●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use.

(STERILIZATION)

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization. ●When using intraorally, sterilize before each use. ●For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize. ●T&F Hybrid Points: Do not immerse into chlorine disinfectant. ●Dura-White Stones: Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. Do not perform plasma sterilization. ●CompoSite and Ceramist: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization. ●Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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