
BeautiBond **X** treme

SHOFU

# STEP CARD




### Adhesive resin cementation for indirect restorations

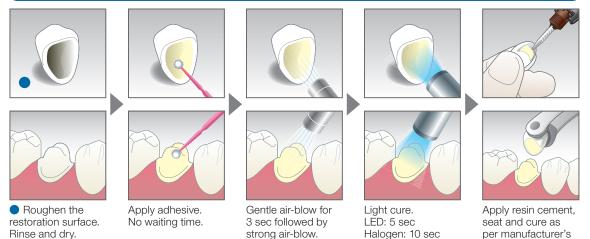

### Immediate dentin sealing for indirect restorations




Prepare cavity or abutment, rinse and dry.

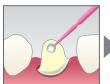

Apply adhesive. No waiting time.



Gentle air-blow for 3 sec followed by strong air-blow.

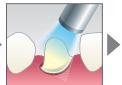

Light cure. LED: 5 sec Halogen: 10 sec



instruction.


Remove unpolymerized layer with alcohol swab.

BeautiBond X treme




## STEP CARD

### Direct restoration with composite resin

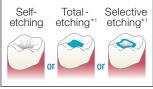

SHOFU

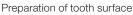

\*1 Follow the standard method for acid etch. Then rinse and dry.



Apply adhesive\*2. No waiting time.

\*<sup>2</sup> Rub the adhesive for 20 sec on phosphoric acid etched tooth surface.



Light cure. LED: 5 sec Halogen: 10 sec



Restore with composite resin.

### Intraoral repair



Roughen surface, rinse and dry.




Apply adhesive. No waiting time.



3 sec followed by

strong air-blow.

Gentle air-blow for 3 sec followed by strong air-blow.

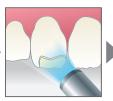

Light cure. LED: 5 sec Halogen: 10 sec



Restore with composite resin.


### Core build-up



Prepare post space and post as per manufacturer's instruction.




Apply adhesive. No waiting time.

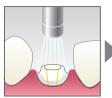

Gentle air-blow for 3 sec followed by strong air-blow.



Light cure. LED: 5 sec Halogen: 10 sec

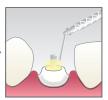

Post cementation and core build-up as per manufacturer's instruction.