



# BEAUTIFIL Flow Plus X

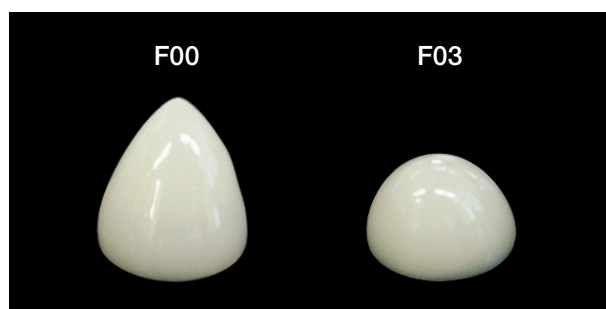

## Injectable hybrid composite in two viscosities

This durable composite restorative for both anterior and posterior teeth allows you to place direct restorations with an excellent self-levelling ability and outstanding physical properties, absolutely equivalent to those of conventional hybrid composites. Thanks to its nano S-PRG filler, Beautifil Flow Plus X not only helps to remineralise the adjacent tooth structure for sustainable caries prevention, but also easily polishes to a high and long-lasting lustre. It comes in two different viscosities, F00 and F03, for restorative build-up and filling.

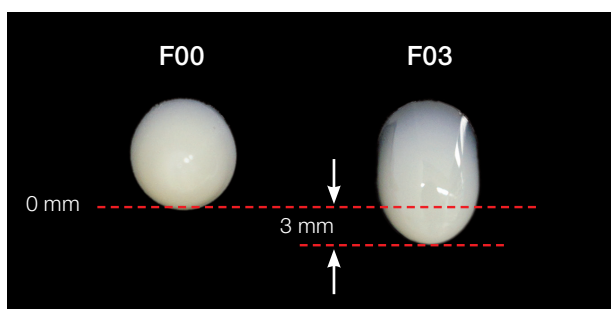

These illustrations show the difference in flow behaviour between the two materials when placed on a vertical (left) or horizontal (right) mixing slab for one minute.

### Indications

#### ■ F00 – ZERO FLOW

Excellent shape retention, ideal for build-up:  
The flowability of F00 is so low that the paste is not deformed during layering. This easy-to-handle composite helps you to accurately restore the occlusal morphology, including cusps and marginal ridges. Main benefit: It flows well during application, but does not slump after sculpting.

#### ■ F03 – LOW FLOW

Moderate shape retention and viscosity plus perfect adaptation: The self-levelling F03 paste thoroughly wets the cavity walls and is therefore ideal for filling. It can be used to line cavity floors, fill small occlusal cavities, or create cervical restorations.

## BENEFITS

- Complete system for all conservative/restorative indications
- Simplified handling in direct applications
- Compatibility with all conventional bonding systems
- Wide range of shades for aesthetic restorations
- Optimal polishability and durable gloss

### Restoring the occlusal surface of a molar



Cavity preparation



Cavity lining with F03



Build-up of cusp ridges with F00



Occlusal morphology after filling



Postoperative result



F00 – absolutely stable (Zero Flow)

Shade	PN 2.2 g	Shade	PN 0.21 g
SYRINGE		TIPS, 20 pcs.	
A1	YE2301	A1	Y2401
A2	YE2302	A2	Y2402
A3	YE2303	A3	Y2403
A3.5	YE2304	A3.5	Y2404
A4	YE2305	A4	Y2405
B1	YE2306	B1	Y2406
B2	YE2307	B2	Y2407
C2	YE2308	C2	Y2408
D2	YE2309	D2	Y2409
SA0.5	YE2300	SA0.5	Y2400
SA0.50 (opacious A0.5)	YE2310	SA0.50 (opacious A0.5)	Y2410
A10 (opacious A1)	YE2311	A10 (opacious A1)	Y2411
A20 (opacious A2)	YE2312	A20 (opacious A2)	Y2412
Inc (Incisal)	YE2314	Inc (Incisal)	Y2414
BW (Bleaching White)	YE2315	BW (Bleaching White)	Y2415



F03 – moderate flow (Low Flow)

Shade	PN 2.2 g	Shade	PN 0.21 g
SYRINGE		TIPS, 20 pcs.	
A1	YE2321	A1	Y2421
A2	YE2322	A2	Y2422
A3	YE2323	A3	Y2423
A3.5	YE2324	A3.5	Y2424
A4	YE2325	A4	Y2425
B1	YE2326	B1	Y2426
B2	YE2327	B2	Y2427
C2	YE2328	C2	Y2428
D2	YE2329	D2	Y2429
SA0.5	YE2320	SA0.5	Y2420
A20 (opacious A2)	YE2332	A20 (opacious A2)	Y2432
A30 (opacious A3)	YE2333	A30 (opacious A3)	Y2433
Inc (Incisal)	YE2334	Inc (Incisal)	Y2434
BW (Bleaching White)	YE2335	BW (Bleaching White)	Y2435
MI (Milky)	YE2336	MI (Milky)	Y2436
CV (Cervical)	YE2337	CV (Cervical)	Y2450



PN	Quantity	Product
2000X	1 pc.	<b>Beautifil Flow Plus X Standard Kit</b> 1 syringe each at 2.2 g: Beautifil Flow Plus X F00 and F03, shade A2 and Beautifil Flow Plus X F00, shade A3
YE2303S	1 pc.	<b>Beautifil Flow Plus X 4 Syringes Kit F00</b> 2 syringes each at 2.2 g Beautifil Flow Plus X F00, shade A2 and A3
YE2323S	1 pc.	<b>Beautifil Flow Plus X 4 Syringes Kit F03</b> 2 syringes each at 2.2 g Beautifil Flow Plus X F03, shade A2 and A3
YE2300- YE2337	2.2 g	Beautifil Flow Plus X - syringe
YE2400- YE2450	0.21 g	Beautifil Flow Plus X - tips, 20 pcs.
Y2351	50 pcs.	Needle tips 20 G-L for Beautifil Flow Plus X